



Clonal Technologies


# Interpretation of Results

## NEGATIVE

A coloured line in the control line region (C) and a coloured line in the test region (T) for a specific drug indicates a negative result. This indicates that the drug concentration in the urine specimen is below the designated cut-off level for that specific drug.

NOTE: The shade of colour in the test region (T) may vary, but it should be considered negative whenever there is even a faint coloured line.

NEGATIVE

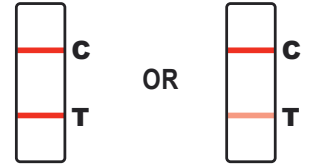

## NON-NEGATIVE

A coloured line in the control line region (C) but no line in the test line region (T) for a specific drug indicates a non-negative result. This indicates drug concentration in the urine specimen exceeds the designated cut-off for that specific drug.

NON-NEGATIVE



## INVALID

Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control failure. Review the procedure and repeat the test using a new test cup. If the problem persists, discontinue using the lot immediately and contact the distributor.

INVALID

